



HEALTHCARE SUPPLY®

7733CRJ/ 7734CRJ

Crest Replacement Station for use with Edwards Lifeline Nurse Call Systems

Connect station as shown below:


- Red wire to System +
- White wire to System -
- Black wire to Corridor Pilot (Switched Output)
- Green wire is an additional switched output. If Green wire is not used it is recommended that it be capped off with a wire nut or cut off flush at the PCB.

7733CRJ uses a 1-gang back box.

7734CRJ uses a 2-gang back box.

Call jacks are intended for use with a momentary 2-conductor ¼" phone plug style call cord – 9900W-XX or equivalent recommended.

Insert dummy plug(s) into unused call jacks when not in use.

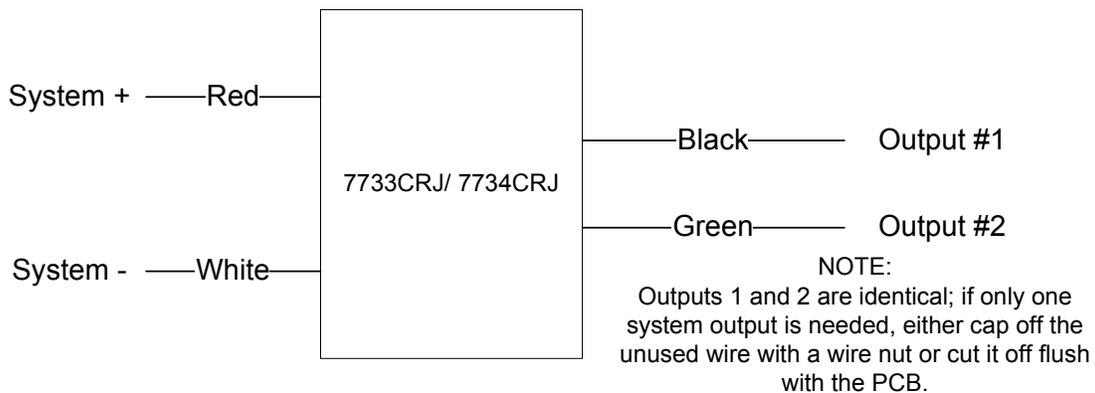

If you have any questions regarding this information, please contact Crest Technical Support at 1-800-328-8908 for assistance.

Crest Healthcare Supply, P.O. Box 727, 195 S. Third St., Dassel, MN 55325
(320) 275-3382 Phone (320) 275-2306 FAX