

Single Patient & Shower Station, Crest Replacement for Dukane®

9A2245C, 9A2240C
9A2206CC, 9A2206C



Technical Specifications

Current & Voltage Rating: 24 VDC @ 20mA
Terminals: Wire Leads
Mounting: Screw to standard single gang wall box
Plate: 4.5" x 2.75" (1-gang)



9A2245C

9A2240C

Typical Installation

1. Replace existing station with Crest replacement, referencing the wire color comparison table on right.
2. When installing the Shower Station place the o-rings on the mounting screws and center the gasket between the faceplate and the wall to ensure a waterproof seal.
3. Always test all nurse call stations for correct operation (including cord out if applicable) before placing into service.

Wire Color Comparison Table

Dukane	Crest
Blue/Black	Yellow
Blue	Blue
Blue/White	Green
Yellow/Black	Orange
Violet	Violet
Violet/Red	Grey
White/Red	White
Violet/Green	Brown
Red	Red
Black	Black



9A2206CC

9A2206C

Crest Technical Support: **1-800-328-8908**

Specifications subject to change without notice.
Visit CrestHealthcare.com for additional technical documents.