
VERIDIAN
HEALTHCARE®

Self-Taking Home Blood Pressure Kit


Model # 01-5501

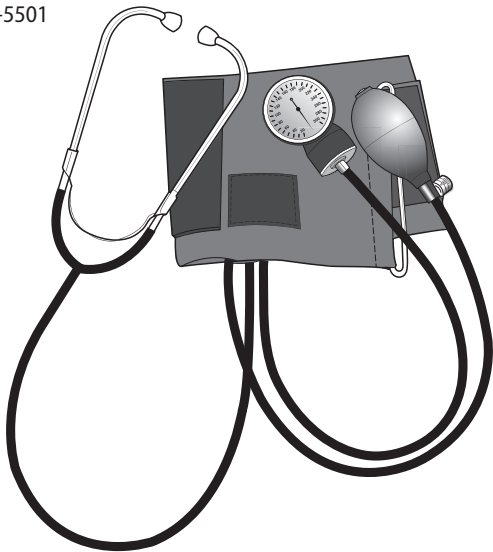**Please read this instruction manual
completely before operating this unit.**

INSTRUCTION MANUAL • ENGLISH & ESPAÑOL

STOP!
PLEASE ENSURE YOU HAVE ALL OF THE FOLLOWING
COMPONENTS BEFORE USING YOUR HOME BLOOD
PRESSURE KIT.

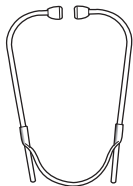Binaural

Gauge


Squeeze Bulb

Adult Arm Cuff
with attached
Stethoscope
and Tubing

Canvas
Storage Case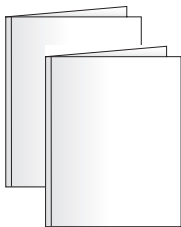

Instruction Manual
Blood Pressure Reading Tracking Log
FAQ Information Sheet
Replacement Parts Order Form

IF YOU ARE MISSING ANY PARTS, INCLUDING INSERTS OR
INSTRUCTION MANUALS, DO NOT RETURN TO PLACE OF
PURCHASE. CONTACT CUSTOMER CARE AT 866-326-1313.

INDEX

Introduction.....	4
Getting Started.....	5
Features and Assembly of Your Home Kit.....	6
Applying Your Blood Pressure Cuff.....	7
Proper Stethoscope Use and Fit.....	8
Taking Your Blood Pressure Reading	9-10
Care and Maintenance	10
Warranty	11
Instrucciones en Español.....	13-24

Toll-Free Customer Care Help Line:
1-866-326-1313
Monday – Friday 8:30 a.m. – 4:30 p.m. CST


Manufactured for
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031

Made in China

#93-1062 01/20
©2020 Veridian Healthcare, LLC

INTRODUCTION

It is recommended that you first seek the advice and recommendation of your physician or healthcare professional when using home diagnostic devices, including blood pressure monitors.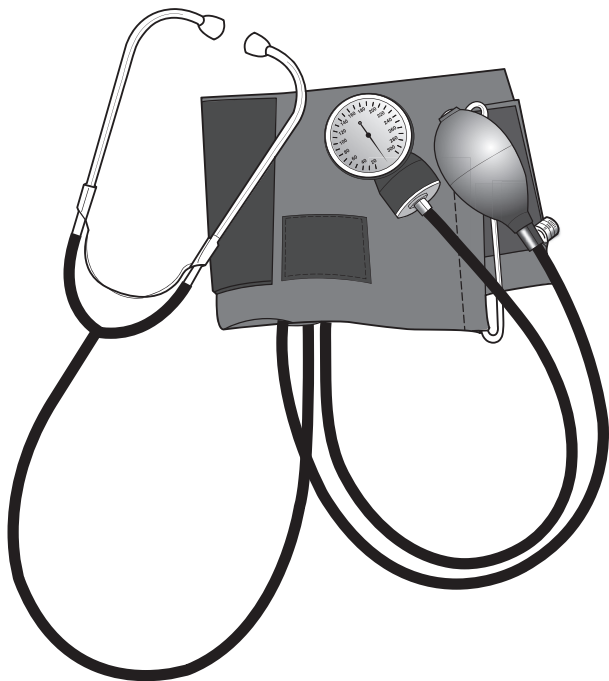


Thank you for purchasing a Self-Taking Home Blood Pressure Kit. With proper care and use, your monitor will provide you with many years of reliable measurements.

GETTING STARTED

Before taking a measurement

1. Your readings should only be interpreted by your physician or healthcare professional with access to your individual medical history. Regular use of a home blood pressure monitor will allow you to track and record your readings for discussion with your physician.
2. Conduct your measurement in a quiet place while seated in a relaxed position. Rest for 15 minutes before taking your reading. If an error occurs or you desire to take a second reading, allow 15 minutes between readings for your blood vessels to return to normal.
3. Avoid smoking, eating, taking medication, alcohol, physical activity or any other stressful activity for 30 minutes prior to taking a reading.
4. Always remove any jewelry or constrictive clothing that may interfere with the cuff placement.
5. Keep yourself and the monitor still during measuring; do not talk during the reading.
6. It is recommended that you take your readings at the same time each day to better monitor any indications in your results.
7. Record your daily measurement on the included chart or some other written document to share with your physician.

FEATURES AND ASSEMBLY

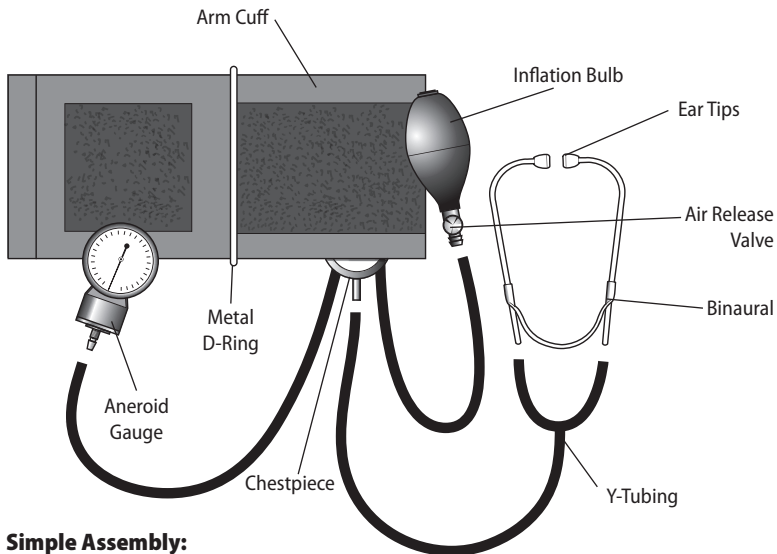Simple Assembly:

1. Insert the single end of the “Y” tubing into the stem of the chestpiece to form a stethoscope.
2. Insert the metal binaural ends into the “Y” tubing.
3. Attach the gauge to the tubing to the left of the stethoscope.
4. Attach the inflation bulb to the tubing to the right of the stethoscope.

When assembled, the white artery patch should be on the outside and the chestpiece should be on the inside of the cuff. If folded properly, the hook and loop material will be on the outside of the cuff.

APPLYING YOUR CUFF

Before applying your blood pressure cuff, be sure you have selected the appropriate size cuff:

Adult Cuff fits arm circumference 10" - 14" (25.4-35.5 cm)

Additional cuff sizes are available for order on the enclosed order form.

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

If for any reason you are unable to or should not use your left arm, please modify the instructions for cuff application to your right arm. Your physician can help you identify which arm is best for you to take measurements from.

1. Remove any constrictive clothing or jewelry that may interfere with cuff placement.
2. Be seated at a table or desk with your feet flat on the floor.
3. Extend your left arm. Find your brachial artery, which is at the inner aspect (bend) of your elbow, Fig 1. You may be able to feel your pulse.
4. Open the cuff creating a circle that you can slide your arm into; do not extend the cuff beyond the metal bar.
5. Insert your left arm into the cuff. Position the bottom edge of the cuff approximately one-half inch ABOVE your elbow; when taking your reading, the chestpiece must directly touch the inside of your arm, over your brachial artery.
6. With your arm extended, palm facing upward and tubes positioned down the center of your arm, Fig 2, use your right hand to pull the end of the cuff and secure the hook and loop material. Make sure that the chestpiece is still touching your brachial artery, has not shifted and is one-half inch above the elbow, Fig 3.
7. The cuff should fit comfortably, yet snugly around your arm. You should be able to insert one finger easily between your arm and the cuff.

Fig. 1

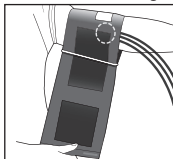
Fig. 2

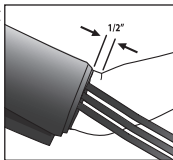
Fig. 3

If you are not comfortable with applying your cuff, please seek the assistance of another member of your household or work with your physician to practice the cuff application. Incorrectly applied cuffs may result in inaccurate readings.

PROPER STETHOSCOPE USE AND FIT

The earpieces on your Self-Taking Home Blood Pressure Kit stethoscope have been carefully calibrated to provide clear and audible acoustics. To best hear the sounds to be analyzed, please keep the following tips in mind:

The earpieces on your Self-Taking Home Blood Pressure Kit stethoscope have been carefully calibrated to provide clear and audible acoustics. To best hear the sounds to be analyzed, please keep the following tips in mind:

The earpieces should point forward when in use, Fig1. The earpieces should fit in your ears securely but comfortably.

The binaurals may be adjusted to ensure maximum comfort and efficacy. To increase the tension, simply squeeze the binaurals just above the joint where they meet, crossing the earpieces, Fig 2. Do not apply too much pressure; apply a minimum of force, slightly increasing the pressure until the earpieces fit comfortably.

Should the earpieces be too tight, pull the binaurals apart, grasping them just above the joint where they meet, Fig. 3. Be gentle, so as not to damage the stethoscope.

Always use caution when adjusting the binaurals to avoid damage. When bringing the earpieces closer together, the maximum tension used should bring the black tubing on either side within contact. When pulling the binaurals apart, do not exceed 90 degrees between the binaurals.

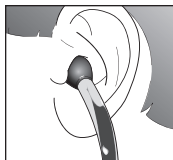
Fig. 1 -
Correct Position

Fig. 1 -
Incorrect Position

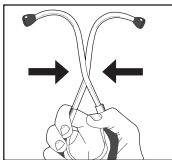
Fig. 2

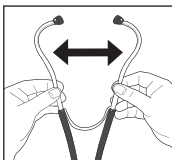
Fig. 3

TAKING YOUR READING

Before taking your reading, review the section under Features & Assembly of your Home Blood Pressure Kit and ensure every piece is properly assembled and your eartips are securely fastened.

Reminder, it is important to avoid smoking, eating, taking medication, alcohol consumption or physical activity 30 minutes prior to taking a reading.

1. Insert the eartips into your ears.
2. Rest your elbow on a table with your palm and arm facing upward. Elevate your hand so that the wrist is at the same level as your heart, Fig 1. Relax your left hand.
3. Position the gauge so it can be read without being moved, such as clipped to a thick notebook or folded towel.
4. Hold the inflation bulb with your right hand and turn the air release valve clockwise to close, Fig 2.
5. Begin inflating the cuff slowly and steadily by pumping the inflation bulb until the gauge reads 30 mmHG ABOVE your last known SYSTOLIC measurement. If you do not know your last known SYSTOLIC measurement, inflate to approximately 180-200 mmHg. NEVER INFLATE THE CUFF ABOVE 280 mmHg.
6. When you have reached the proper inflation level, slowly and steadily turn the air valve counter-clockwise at a rate of 2-3 mmHg per second. You will be able to watch the progress of the release on the gauge.
7. As the pressure drops, listen closely for your pulse beat while watching the gauge. Note the first thumping sound you hear, this is your SYSTOLIC pressure, Fig 3.

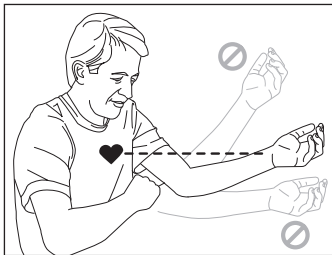
Fig. 1

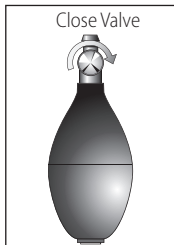
Fig. 2

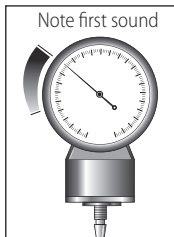
Fig. 3

TAKING YOUR READING

8. Allow the cuff to continue to deflate. You will hear changes in the sounds and then no sound at all. When you have heard the last sound, note the reading measurement, this is your DIASTOLIC pressure, Fig 4.
9. Quickly turn the valve counter-clockwise to deflate the remaining air, Fig 5. Remove the cuff and record your measurement on the included recording chart.

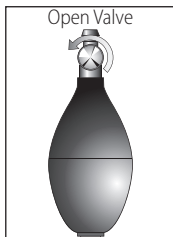
Fig. 4

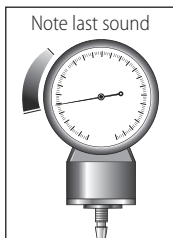
Fig. 5

IF YOU WISH TO TAKE ANOTHER MEASUREMENT, WAIT A MINIMUM OF 15 MINUTES BETWEEN READINGS TO ALLOW FOR THE BLOOD VESSELS TO RETURN TO NORMAL.

CARE & MAINTENANCE

- Do not use any liquids on the gauge or cuff
- Do not store in direct sunlight, dust or excessive humidity
- Avoid extreme temperatures
- Do not disassemble the gauge or cuff
- Avoid dropping or hitting the surface of the gauge or stethoscope chestpiece

When the cuff is fully deflated, make sure that the needle on the gauge is within the zero square range. If the needle is outside of the zero range, it needs to be recalibrated by a qualified technician. Ship your gauge, prepaid, to Repair Department, 1175 Lakeside Drive, Gurnee, IL 60031.

1-YEAR LIMITED WARRANTY

The warrantor guarantees that its Self-Taking Home Blood Pressure Kit will be free from manufacturing defects under normal use for a period of one-year. This warranty covers only normal use, and does not apply to use in any clinical or commercial applications. This warranty is voided if the unit is misused or abused in any manner.

If the Self-Taking Home Blood Pressure Kit fails to operate within the warranty period while under the original purchaser owns the product, return it postage prepaid to: **Veridian Healthcare, Attn: Repair Department, 1175 Lakeside Drive, Gurnee, IL, 60031**; include your name, address, phone number and brief description of the problem with your product. Take care to carefully package the product to avoid any damage that may occur while in transit; shipping insurance with returned receipt is recommended. Veridian Healthcare will repair or replace the defective unit at our option. Repair or replacement of the defective unit, at the warrantor's option, is the sole remedy under this warranty.

ANY IMPLIED WARRANTIES WHICH THE PURCHASER MAY HAVE ARE LIMITED IN DURATION TO THE TIME THAT THE ORIGINAL CONSUMER PURCHASER OWNS THE PRODUCT. Some states do not allow limitations or how long an implied warranty lasts, so the above limitation may not apply to you.

This warranty constitutes the warrantor's only reasonability and obligation to repair and/or replace materials or components, or refund the purchase price. Veridian will not be responsible for any indirect, incidental, special consequential or punitive damages or other loss, including, but not limited to damage to or loss of other property or liable to the purchaser for any amount in excess of the cost of repair and/or replacement of the unit, or the purchase price of the unit. Some states do not allow the exclusive or limitation of incidental or consequential damages, so the above limitation or exclusion may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

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