Premium Pulse Oximeter Tracking Software

For use with Pulse Oximeter Models
Model 11-50DP

The Premium Pulse Oximeter Tracking Software allows you to real-time monitor pulse oxygen measurements, save those measurements and store previously recorded measurement sessions from your device.

This software is can be downloaded at:

www.veridianhealthcare.com/product-support/

SYSTEM REQUIREMENTS:

- IBM Compatible
- Windows XP, VISTA, 7, 8, 10
- USB Communications Port
- 16MB Memory
- 10MB Free Space on Hard Drive

SOFTWARE INSTALLATION

Visit :

www.veridianhealthcare.com/product-support and follow the on-screen instructions to install your software.

The following icons will appear as shortcuts on the desktop:



Select  for real-time monitoring;

Select  to download a saved recording from the device.

SPO2 REAL-TIME MONITORING

1. Connect the USB cable to an open port on your PC and to the Pulse Oximeter.
2. Launch the SpO2 software. Follow the onscreen prompts to select the desired language.
3. Click the  button in the main window interface. The dialog box of the device connection will appear after doing the above operation as figure: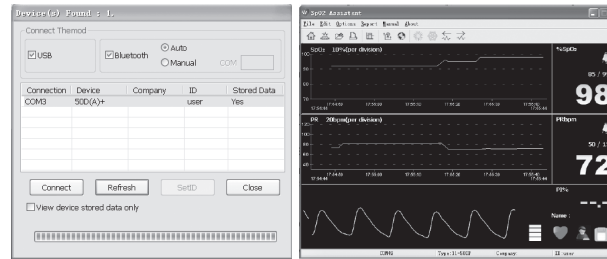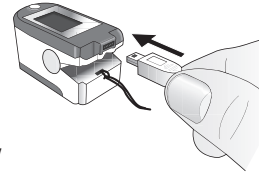

In connection mode selection area, select the connection mode of software and device, after selected, click "Refresh" button to start the connection.

REVIEW OF SAVED MEASUREMENTS


The saved measurement data from real-time monitoring and recorded measurements can be saved and stored on your PC.

To review the measurements:

1. Connect the USB cable to an open port on your PC and to the Pulse Oximeter.
2. Launch the **SpO2 review** software.
3. Click on the **magnifying glass** icon, then select the review button at the top of the screen.
4. Select the desired measurement for review.

SPO2 REVIEW OF RECORDED MEASUREMENTS FROM PULSE OXIMETER

1. In real time mode, click "File-Device Stored Data" and the device stored data dialog box will appear.
2. Select the data to be uploaded. Choose the desired user, then click the receive data button. A list of stored data will then appear. Click the cancel button if you wish to stop the upload.

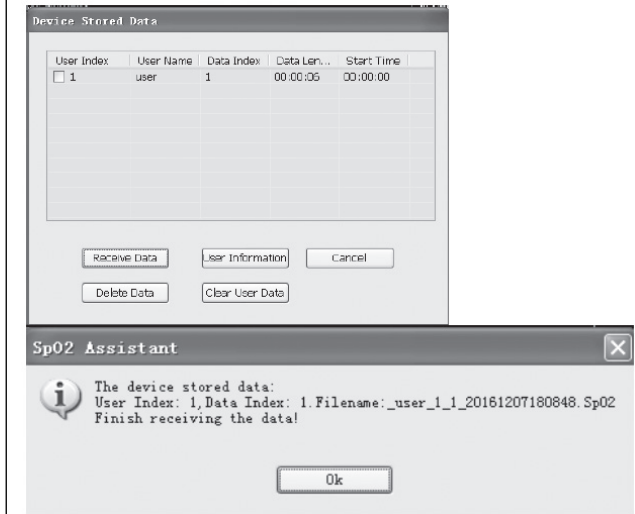PRINTING SAVED MEASUREMENTS

Reports are available for printing: Summary, Full Study, Oximetry and Strip Chart. Use the printer icon on the bottom right to print the desired report following the on-screen prompts.

IF YOU HAVE ANY QUESTIONS OR ARE EXPERIENCING DIFFICULTIES, PLEASE CALL THE TOLL-FREE CUSTOMER SERVICE HELP LINE FOR FURTHER ASSISTANCE AT 1-866-326-1313.

Distributed by:
Veridian Healthcare
1175 Lakeside Drive
Gurnee, IL 60031
www.veridianhealthcare.com
Made in China
#93-8007 07/19
©2019 Veridian Healthcare, LLC

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www.veridianhealthcare.com/product-support

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