8"X11" 614-0080-9812

COMPRESA DE GEL REUTILIZABLE

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REUSABLE HOT & COLD GEL COMPRESS

Microwaveable
Non-Toxic, No Dyes Added
Use only as directed, do not apply directly to skin
For external use only

DIRECTIONS FOR USE:

Always inspect compress for cracks or tears
before use.

HOT COMPRESS:

Hot Water Heating Method:

1. Place the compress in boiling water for 2 minutes, or until
heated thoroughly.
2. Remove compress from water with kitchen tongs.
3. Wrap the compress in a dry or moist towel and apply to treatment area.

Microwave Heating Method:

1. Place compress in the microwave and heat at full power for
50 seconds. Note that microwave heating times and power may vary.
2. Remove compress from microwave. If additional heat is needed
increase the time in 10 second intervals.
3. Wrap the compress in a dry or moist towel and apply to treatment area.

COLD COMPRESS:


1. Place compress in the freezer for a minimum of one hour.
2. Remove compress from freezer, wrap in a dry towel and apply to
treatment area.

Hot Water Heating

Microwave Heating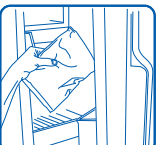

Compress in Freezer

Always Wrap in Cloth

CAUTION:

Always wrap with a towel or cloth before application to treatment area;
do not apply directly to skin. If hot or cold therapy becomes uncomfortable, remove product.
For external use only. Do not use bag if it is punctured, torn or leakage occurs.

WARNING:

Use carefully. May cause serious burns. Do not use over sensitive skin areas or in the presence of
poor circulation. Severe skin burning and possible permanent injury may occur if skin temperatures reach
above 107° F (41.6° C); extreme cold may cause frostbite. The unattended use of Reusable Hot/Kold™ Gel
Compress by children or incapacitated persons may be dangerous. Hot and cold therapy is not recommended
for people with circulatory problems, Raynauds Syndrome or diabetes without consent by your physician.

If gel comes in contact with eyes or open wound, flush with water. If gel comes in contact
with skin, remove any contaminated clothing and wash the area with mild soap and water.
Seek medical attention for any persistent symptoms including irritation or redness.
Do not apply directly to skin.

This product is non-toxic. Discard if bag becomes damaged or leaks.
Do not mix with combustible materials.

Customer Care Help Line
1-877-505-6190

Distributed by:
Duro-Med Industries
Div. of DMS Holdings Inc.
1931 Norman Drive South
Waukegan, IL 60085 USA
www.mabisdmi.com
Made in China

614-0080-9812
0 4 1298 00265 9
#941-0614-0080 01/09
©2009 Duro-Med Industries

pms 185

pms 286