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1.1. Designated use

1.1.1. Intended purpose

Minor surgical luminaire (Treatment light) – light for local illumination of the patient's body during diagnosis or treatment. It is not intended for use in operating theatres.

1.1.2. Location

Only in dry rooms not subject to the risk of explosion. The minimum distance to inflammable parts has to be 0.5m.

1.1.3. Operation mode

The light is designed for continuous operation.

2. Safety instruction

The light may only be operated in dry rooms, **not subject to the risk of explosion.**

Do not leave the light on without supervision.

The minimum distance to inflammable parts has to be 0.5 m.

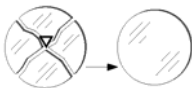
Keep inflammable substances and objects away from the light and the light cone.

When using lights of protection class I make sure that the grounding conductor is connected with the lamp housing.

Do not touch the luminous medium while the light is switched on and for some time afterwards – **danger of burns.**

Defective apertures must be replaced – **Risk of injury.**

Apart from changing the lamps, no further modifications or work may be carried out by the user.

Defective lamps may only be replaced with lamps with the same power (W, Watt) and of the same type (specifications see 'Technical Data' at page 20).

Do not kink the power supply cable to avoid damages.

Do not touch the protective glass.

The user has to comply with the requirements of the national institute for hygiene and disinfection. **Risk: Transmission of diseases.**

3. Abbreviation and Symbols

Attention, observe accompanying documents

Device of international protection class I (protective earth)

I

ON

0

OFF

I

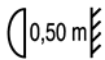
ON (stage I = 60%)

II

ON (stage II = 100%)

Attention! Hot surface

Keep the minimum distance of 0.5m to inflammable parts

Defective shades must be replaced

Symbol for separate disposal according the regulation 2002/96/EC (WEEE - Waste Electrical and Electronic Equipment)

UL-Conformity mark (USA + Canada)

VDE-Conformity mark

CE-Conformity mark

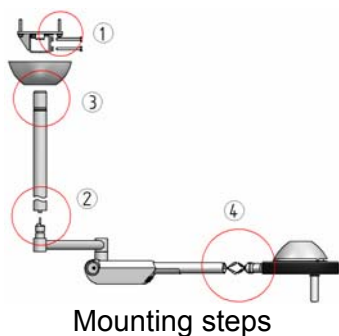
The manufacturer cannot be held liable for damages caused if the light is used for other purposes than the intended purpose or if safety instruction and warnings are ignored.


4. Assembly and Installation 'D^{med}® isis 50 C'

Note: Only qualified technicians should carry out isis installation.

Two people are required for assembling the 'D^{med}® isis 50 C'. The supply voltage must correlate with the voltage noted on the fixture's rating label. The ceiling fitting is to be used exclusively at ceilings that are either solid, or supplied with a suitable high-strength ceiling-mounting solution.

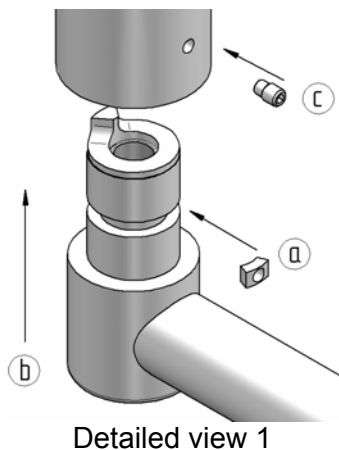- ① Drill holes in accordance to the enclosed drilling template ($\varnothing=8\text{mm}$; depth=55mm). Clean out the drilled holes. Attach the anchor plate using the four anchor bolts. Tap the anchor bolts completely in, using a hammer. Tighten the nut with an open-ended wrench (approx. 3-4turns, 23Nm torque).


- ② Perform this Step on a flat and level surface. Pull the cable through the support tube. Insert the slider into the groove of the arm joint, noting the location of the fixing hole (see detailed view 1 part 'a'). Slide the arm and support tube together (view 1, direction 'b'). Align the hole in the slider with the set-screw hole at the bottom of the support tube. Insert the set-screw into the hole to lock the slider in place (1c). Tighten the set-screw completely, then loosen it by a maximum 1-quarter turn and check the operation of the arm joint.
Check: **Set-screw has to be flush.**

- ③ Place the O-ring and the ceiling cover over the support tube. Insert the support tube into the anchor plate and fix it temporarily, using supplied bolts (see detailed view 2). Align the support tube to the direction of the fixture's general usage. Tighten the screws on the anchor plate. Now connect to the electrical power supply, as per the wiring diagram (see sheet enclosed with the set of bolts). Slide the ceiling cover and the O-ring upwards.

- ④ Head Assembly (see page 17)

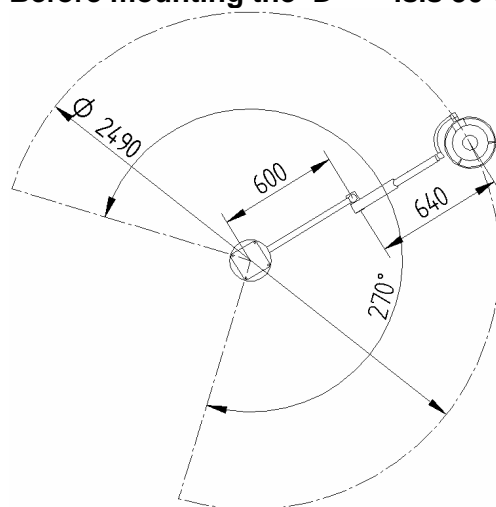
Before mounting the 'D^{med}® isis 50 C', take heed of the operating range.

5. Assembly and Installation 'D^{med}® isis duo 50/50 C'

Note: Only qualified technicians should carry out isis installation.

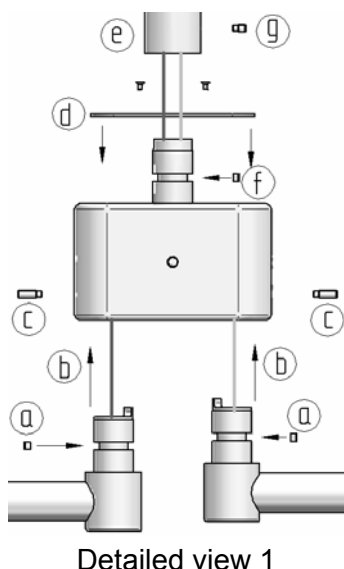Two people are required for assembling the 'D^{med}® isis duo 50/50 C'. The supply voltage must correlate with the voltage noted on the fixture's rating label. The ceiling fitting is to be used exclusively with ceilings that are either solid, or supplied with a suitable high-strength ceiling-mounting solution.

Detailed view 1

- ① Drill holes in accordance with the enclosed drilling template ($\varnothing=8\text{mm}$; depth=55mm). Clean out the drilled holes. Attach the anchor plate using the four anchor bolts. Tap the anchor bolts completely in, using a hammer. Tighten the nut with an open-ended wrench (approx. 3-4turns, 23Nm torque).

- ② Perform this Step on a flat and level surface. Pull the cable through the bearing head. Insert the slider into the groove of the arm joint, noting the location of the fixing hole (1a). Align the hole in the slider with the set-screw hole at the bottom of the bearing head. Insert the set-screw (18 mm) into the hole to lock the slider in place (1c). Tighten the set-screw completely, then loosen it by a maximum 1-quarter turn and check the operation of the arm joint.

Check: **Set-screws had to be flush.**

Pull the cable through the cover (1d) and the support tube (1e). Insert the slider into the groove of the arm joint, noting the location of the fixing hole (1f). Slide the support tube and the bearing head together (1e). Align the hole in the slider with the set-screw hole at the bottom of the support tube. Insert the set-screw (10 mm) into the hole to lock the slider in place (1g). Tighten the set-screw completely, then loosen it by a maximum 1-quarter turn and check the operation of the arm joint. Fix the cover (1d) with the four screw (1h)

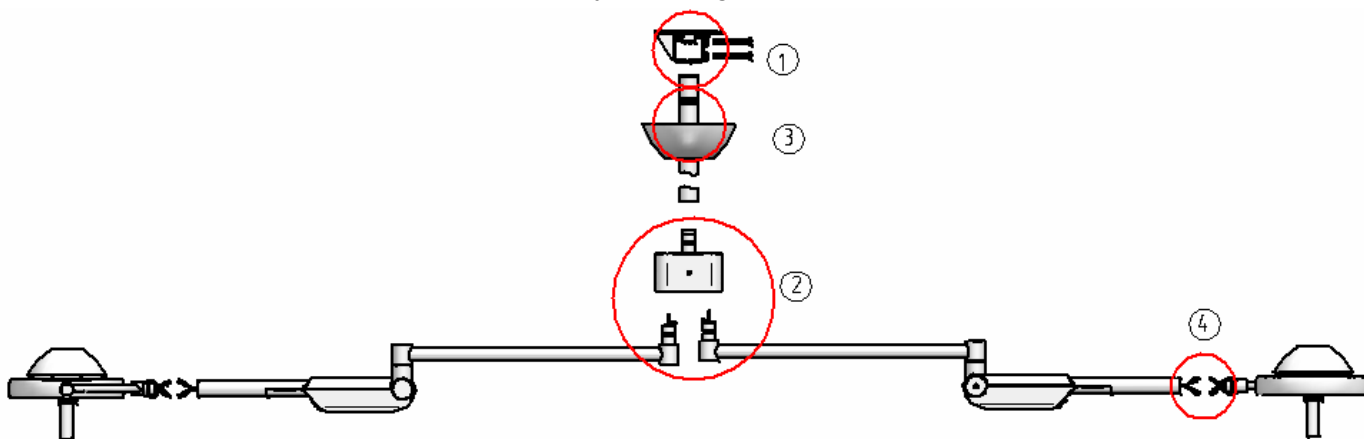
Check: **Set-screw has to be flush.**

Detailed view 2

- ③ Place the O-ring and the ceiling cover over the support tube. Insert the support tube into the anchor plate and fix it temporarily, using supplied bolts (see detailed view 2). Align the support tube to the direction of the fixture's general usage. Tighten the screws on the anchor plate. Now connect to the electrical power supply, as per the wiring diagram (see sheet enclosed with the set of bolts). Slide the ceiling cover and the O-ring upwards.

- ④ Head Assembly (see page 17)


Mounting steps

6. Assembly and Installation 'D^{med} isis 50 W'


Note: Only qualified technicians should carry out isis installation.

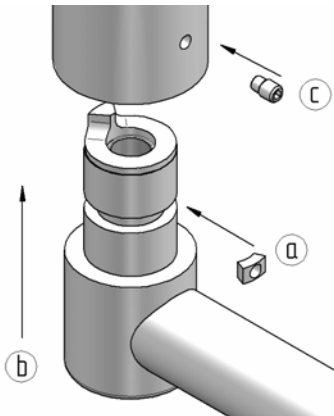Two people are required for assembling the 'D^{med} isis 50 W'. The supply voltage must correlate with the voltage noted on the fixture's rating label. The wall fitting is to be used exclusively with walls that are either solid, or supplied with a suitable high-strength wall-mounting solution.

Detailed view 1

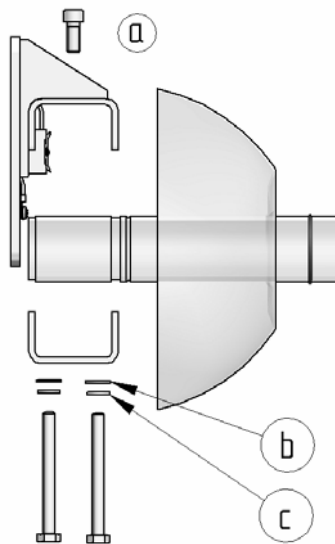
- ① Drill holes in accordance with the enclosed drilling template ($\varnothing=8\text{mm}$; depth=55mm). Clean out the drilled holes. Attach the anchor plate using the four anchor bolts according to detailed view 2. Tap the anchor bolts completely in, using a hammer. Tighten the nut with an open-ended wrench (approx. 3-4turns, 23Nm torque).
- ② Perform this Step on a flat and level surface. Pull the cable through the support tube. Insert the slider into the groove of the arm joint, noting the location of the fixing hole (see detailed view 1 part 'a'). Slide the arm and support tube together (view 1, direction 'b'). Align the hole in the slider with the set-screw hole at the bottom of the support tube. Insert the set-screw into the hole to lock the slider in place (1c). Tighten the set-screw completely, then loosen it by a maximum 1-quarter turn and check the operation of the arm joint.


Detailed view 2

- ③ Place the O-ring and the ceiling cover over the support tube. Insert the support tube into the anchor plate and install the locking bolt (Detailed view 2a) from top. Tighten the screws on the anchor plate. Observe the correct order of the washer (2b) and the serrated lock washer (2c). Now connect to the electrical power supply, as per the wiring diagram (see sheet enclosed with the set of bolts). Slide the ceiling cover and the O-ring to the anchor plate.

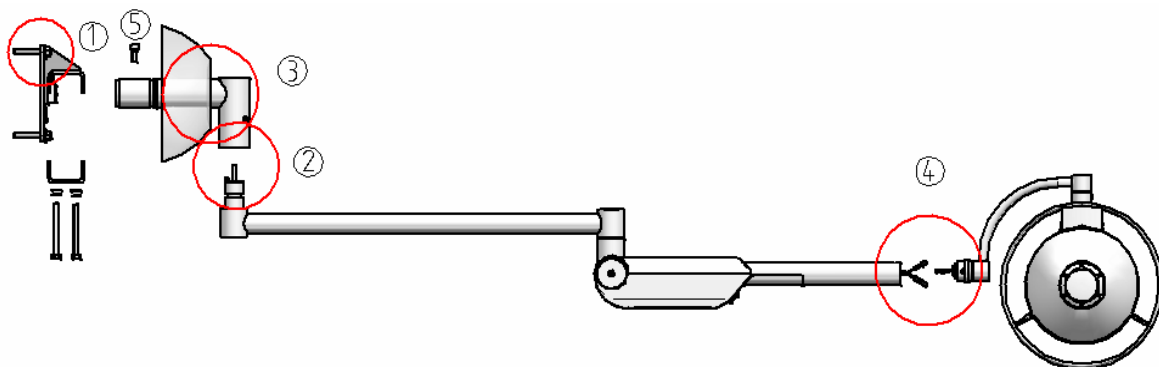- ④ Head Assembly (see page 17)


Mounting steps

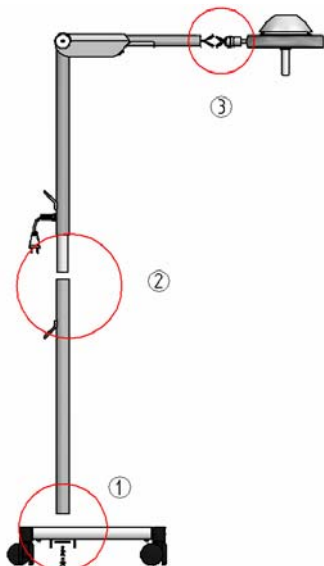Mounting steps


7. Assembly and Installation 'D^{med}® isis 50 F'

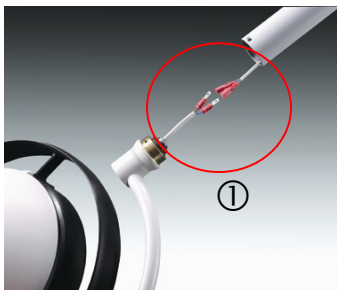
- ① Insert the support tube into the castor base and tighten the set-screws (support disc, screws - 1xM8, 1 x M5 – and serrated washer).
- ② Place the spring-loaded arm on the support tube and align the holes. Attach the cable holder with the supplied screws and tighten.
- ③ Head Assembly (see page 17)

8. Head Assembly

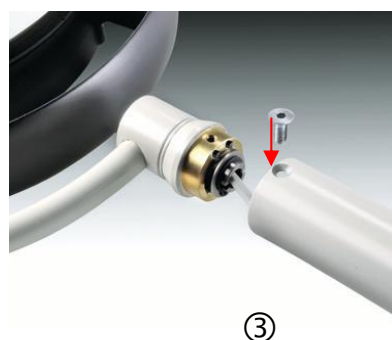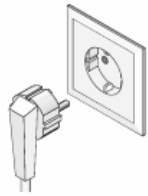Match the plug- and socket connectors as shown in Fig ① completely. The active parts has to be total covered. Align the hole on the end of the top arm with the hole on the limiter and slide the limiter into the top arm (see Figs. ② and ③). Insert and tighten the two set-screws to hold the head in place (see Fig. ④).

9. Putting into Operation

Before placing the light into operation, the user has to check whether mains voltage and frequency specified on the rating plate are identical.

10. Care, Maintenance and Repair

Disconnect the light from the mains before carrying out any maintenance or repair work!

This work must be carried out only by a qualified electrician!

As replacement parts only use part approved by the manufacturer.

10.1. Regular checks

The mains line must be checked minimum once a a year for damage (only 'D^{med}® isis 50 F'). In case of damage the manufacturer must replace the defective mains line.

10.2. Adjustment of the counter-balance (Fig. ①)

Figure ①

The counter-balance arm will already have been factory set for best possible operation. Should adjustment become necessary: Set the spring tension as illustrated, and using a clearance of 4 mm.

10.3. Changing the lamp

Note: Let the halogen lamp cool down before changing it!

①

1. Lift the hinged protective back cover of the head (see Fig. ①)
2. The required Allen wrench is located inside the back cover (see Fig. ②).
3. Remove the lamp head retaining screw (see Fig. ③)
3. Remove the cover ring and reflector
4. Change the halogen lamp (use a soft cloth and do not touch the glass surface of either the old or new lamp, see Fig. ④)
5. Replace the reflector and cover ring. Replace and tighten the screws.
6. Replace the Allen wrench (see Fig. ②)

Note: Only use lamp types approved of by the manufacturer (see 'Technical Data' on page 20).

②

③

④

10.4. Troubleshooting

1. Check the on/off Switch
2. Check the electrical supply
3. Check the halogen lamp, change if necessary
4. Check all terminal connections in the fixture
Also check the voltage on the connecting terminal.

10.5. Cleaning

Note: The periodical cleaning of the face-plate ensure a better light efficiency. Use a usual glass cleaner.

Do not use concentrated disinfectant or dissolver.

For cleaning the outer surfaces, use a cloth moistened with usual cleaner.

Observe the compatibility with varnishes an plastics of the used cleaners.

10.6. Disinfection

According to instruction diluted brands can used for cleaning the outer surfaces:

- ◆ Lysoformin[®]
- ◆ Dismozon[®]
- ◆ Hexaquart[®]plus
- ◆ Sagrotan[®]-Rapid disinfection cleaner

Observe all applicable local regulations.

The disinfection cleaner are applied only for **wipe disinfection**. Spraying disinfection is not allowed.

10.7. Sterilisation

The grip can be sterilised for 30 minutes at 120° and 1.05 bar.

11. Technical Data

Electrical values:

Nominal voltage:	120 V ~
Frequency range	50/60 Hz
Power consumption:	50 W
Integrated electronic transformer	120 V / 12 V

Luminous values:

Central illuminance (0.8 m)	45'000 lux
Central illuminance (1.0 m)	≥ 40'000 lux
Light field diameter 0.8 m D10	∅ = 15cm
Colour temperature	ca. 4000 K
Colour rendering index	Ra 90

Ambient conditions for transport, storage and operating:

Ambient temperature (transport and storage)	-20°C to +70°C
Ambient temperature (operating)	0°C to +35°C
relative humidity	max. 90%

Fitted with:

Halogen lamp	1 x 12 V / 50 W
Type	QT12 50 W

Classification:

Protection class (D ^{med} ® isis50 F)	II
Protection class (D ^{med} ® isis 50 W, D ^{med} ® isis 50 C, D ^{med} ® isis duo 50/50 C)	I
Int. protection according to IEC 529	IP 20
Rating according to 93/42 EWG - Appendix IX	Class I
Safety inspection according	EN 60601-1 IEC 601-1 UL 601-1

12. Disposal

At the end of its service life, the lamp and the halogen lamps must be disposed of in accordance with local regulations.

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Waldmann Lighting Pte. Ltd.
Derungs Licht AG
Derungs Licht AG ist ein Unternehmen der Waldmann Group
Derungs Licht AG, une société du Groupe Waldmann
Derungs Licht AG è un'Azienda del Gruppo Waldmann